




# Dexcom Clarity (app)

Dexcom

## Data sharing with healthcare professionals\*

Dexcom ONE+/G7

Dexcom G6

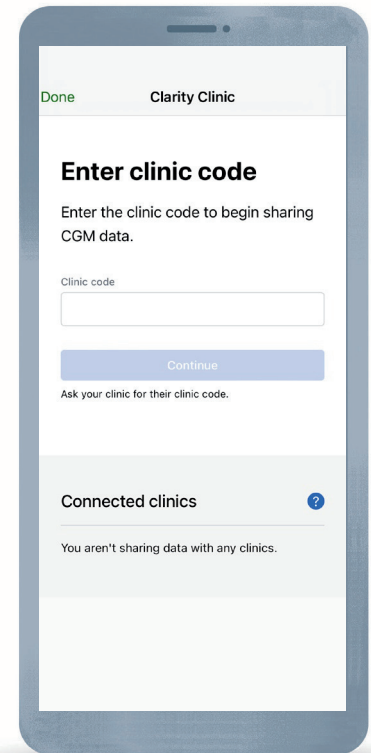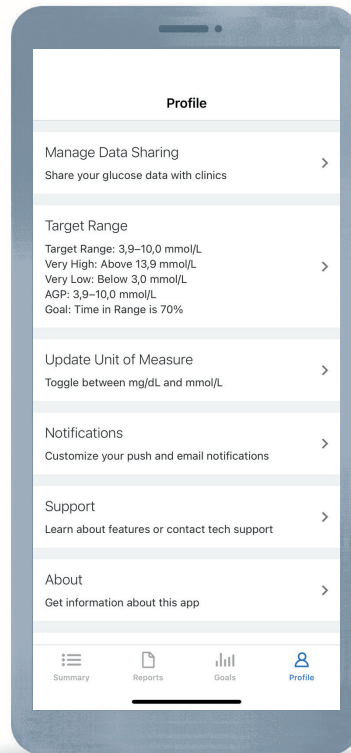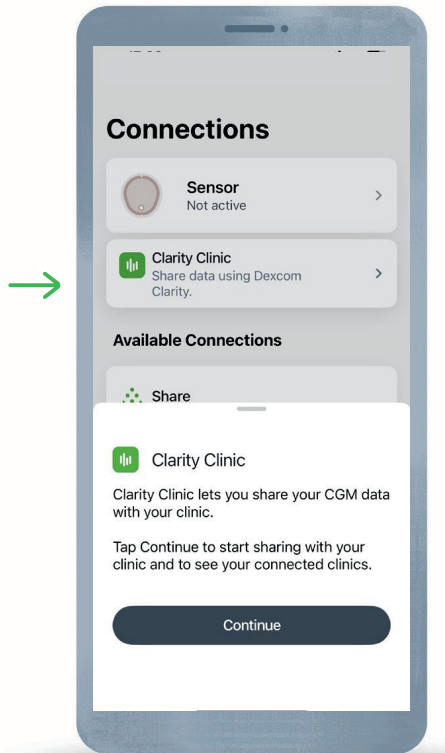

Open the Dexcom ONE+/G7 app.  
Go to **Connections** and tap  
on **Clarity Clinic**.

Open the Dexcom Clarity app.  
Go to **Manage Data Sharing**.

Ask your healthcare  
provider for the clinic code  
and enter.

## Dexcom Clarity

Download the Dexcom Clarity app or visit Dexcom Clarity at: [clarity.dexcom.eu](https://clarity.dexcom.eu) to view detailed reports and statistics of your glucose data.

**Tip:** In the **glucose** tab of your Dexcom ONE+/G7 app, you can already find a Clarity summary with the most relevant statistics.



Log in with the same credentials as your other Dexcom app or create a new account.

\*Patient must first sync their Dexcom CGM data with the Dexcom Clarity app and consent to share their data with their healthcare provider. An internet connection is required for patients to send their glucose data to Dexcom Clarity via a compatible smart device: [dexcom.com/compatibility](https://dexcom.com/compatibility)

Dexcom, Dexcom Share, Dexcom Follow, and Dexcom Clarity are registered trademarks of Dexcom, Inc. in the United States and/or other countries.  
©2025 Dexcom, Inc. All rights reserved. MAT-8758 Rev001