


Dexcom Clarity (receiver)

Data sharing with healthcare professionals*

Your healthcare provider must provide you with the clinic code to start sharing data. Follow the instructions and give the clinic permission to access your data¹.

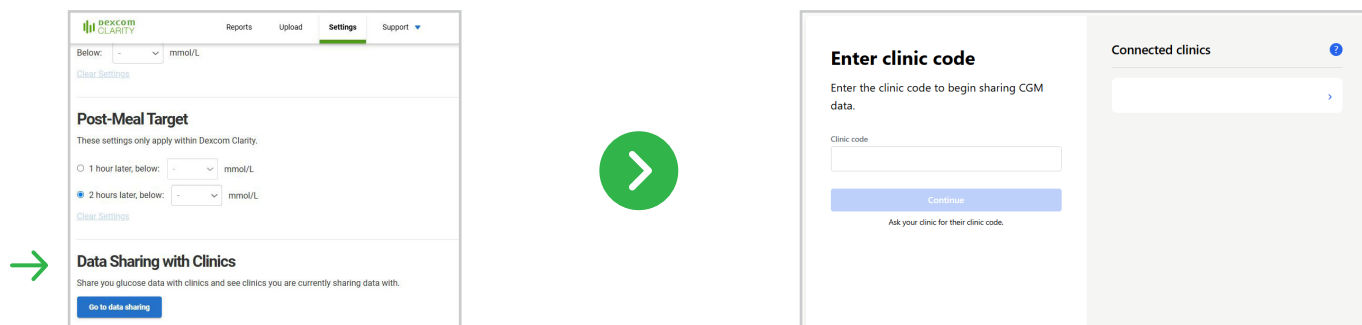

Go to **Settings** and scroll down to **Data Sharing with Clinics**, then click on **Go to data sharing**

Ask the clinic code from the healthcare provider and enter it.

Visit Dexcom Clarity at: **clarity.dexcom.eu** to upload the glucose data from your receiver to your Clarity account, so you can track your progress and share data with your healthcare provider.

Is your healthcare provider going to do this for you?

Take the receiver to the appointment. The healthcare provider can also read the receiver at the healthcare facility.

Select **Dexcom Clarity for home users** and log in with an existing Dexcom-account or create a new account.


1

Go to **upload**.

2

Click on **download** and follow the on-screen instructions to install the software.

Connect your receiver to your computer to upload glucose data from your receiver.

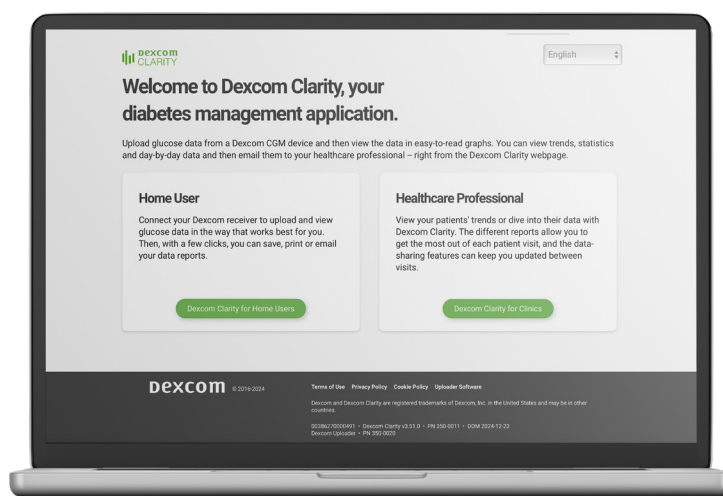

For future uploads, you only need to connect your receiver to your computer.

* Patient must give consent to share their data with their healthcare provider.

¹ An internet connection is required for patients to send their glucose data to Dexcom Clarity via: clarity.dexcom.eu.