

Dexcom G6 to **Dexcom G7**



Actual size



Actual size

TRANSITION GUIDE

Transitioning to a new system can raise questions. So, we've developed this guide to help you navigate Dexcom G7. Dexcom G7 is our most advanced continuous glucose monitoring (CGM) system yet, designed with new upgrades inspired by your feedback. With a sensor that's 60% smaller, it's more discreet, more accurate, and easier to use, making your diabetes management simpler than ever.¹

EASY TO GET STARTED & USE



SIMPLE SETUP

All-in-one sensor and transmitter with guided onboarding and educational tips



30-MINUTE WARMUP

The fastest warmup time out of any CGM brand, with a 12-hour grace period²



ONE-PUSH APPLICATION

92% of surveyed Dexcom G7 users find somewhat easy or very easy to use⁴

ACCURATE¹⁻² AND CONNECTED



EXCEPTIONAL ACCURACY

Our most accurate CGM technology for confident treatment decisions



SEAMLESS CONNECTIVITY

At parity with Dexcom G6—works with leading insulin pump, pen, and health app partners



ENHANCED BLUETOOTH

The Dexcom G7 sensor has a Bluetooth range up to 10 metres for strong and reliable connectivity to your smartphone³

PERSONALISED FOR YOUR NEEDS



NEW ALERT OPTIONS

Customizable alerts, plus new options like Delay 1st High alert and Silent Mode for greater flexibility



DISPLAY FLEXIBILITY

View your glucose numbers across three devices—smartphone, receiver, and smartwatch

Your sensor can send data directly to your Apple watch via Bluetooth (no phone needed).⁵



WATERPROOF

Adapts to your lifestyle with the only waterproof sensor⁶

MAKE THE EASY TRANSITION FROM DEXCOM G6 TO DEXCOM G7 NOW

HOW TO SET UP YOUR DEXCOM G7



If using a smartphone, start here

1. Check the compatibility of your smart device at dexcom.com/compatibility.
2. Download the Dexcom G7 app from the App Store or Google Play.
3. Log in with your existing account and follow the instructions to set up the app.



If using a receiver, start here

1. Ensure you are using a Dexcom G7 receiver.
2. Apply your sensor. An optional overpatch is included to help your sensor remain secure during your wear session.
3. Get your readings after the quick 30-minute warmup. Optional calibration is available for your convenience when starting a new sensor.

TRANSITION TIPS

Can I still connect to my existing insulin pump?

Dexcom G7 is integrated with the latest insulin pumps, pens, and health apps. For the most up-to-date information on compatibility and pairing instructions, please visit your device manufacturer's website. Specific steps will vary depending on your insulin pump, pen, or other connected device.

Can I use my Dexcom G6 receiver with Dexcom G7?

No, Dexcom G6 components are not compatible with Dexcom G7. To get a Dexcom G7 receiver, please contact [\[INFO\]](#).

Is my phone or smartwatch compatible with Dexcom G7?

Visit dexcom.com/compatibility to see if your Apple or Android device is compatible.

Do I need to create a new Dexcom Clarity or Dexcom Follow account?

No, if you are transitioning from Dexcom G6 you do not need a new account. Simply use your existing username and password on all Dexcom apps. Your Followers will also be automatically synced.

NOW WITH MORE FEATURES TO LOVE



GO DISCREET WITH SILENT MODE

Customize alerts to your lifestyle—perfect for those moments when you need to stay focused without interruptions



PERSONALIZE WITH DELAY 1ST HIGH

Fine tune your first high glucose alert to fit your needs, for fewer disturbances and rebound hypos

	Dexcom G6	Dexcom G7
SENSOR		
All-in-one sensor and transmitter	—	✓
Warmup	2 hours	30 minutes
12-hour grace period	—	✓
Water performance	Water resistant	Waterproof ¹
Bluetooth Rapid Connect	—	✓
FEATURES		
Customisable sound and volume	✓	✓
High and low alerts	✓	✓
Urgent Low Soon alert	✓	✓
Delay 1st High alert	—	✓
Silent mode	—	✓
Alert Schedule	✓	✓
COMPATIBILITY		
Standard smart watch mode	✓	✓
Direct to Watch mode ⁵	—	✓
Seamless integration with a selection of insulin pumps, pens, and health partners	✓	✓

We're here to support your transition.

Learn more at <https://www.dexcom.com/nl-nl/g6-transition>.



¹Compared to previous generation, Dexcom G6. ²Compared to other CGM brands. ³Based on a survey in the UK from Dexcom G7 users, N = 231. ⁴Smart devices sold separately. Compatible smartphone is required to pair a new Dexcom G7 sensor with a compatible Apple Watch. To use Share/Follow the smartphone must be within 6 meters. ⁵The Dexcom G7 Sensor is waterproof and may be submerged under 2.5 meters of water for up to 24 hours.

1 Garg SK, et al. *Diabetes Technol Ther*. 2022;24(6):373-380. 2 Laffel LM, et al. *J Diabetes Sci Technol*. 2023;17(4):962-967. 3 Dexcom G7 user Guide. 4 Dexcom Data on File, 2024. 5. FreeStyle Libre and Medtronic user guides.

Dexcom, Dexcom Clarity, Dexcom Follow, Dexcom One, Dexcom Share, Stelo, and any related logos and design marks are either registered trademarks or trademarks of Dexcom, Inc. in the United States and/or other countries. ©2025 Dexcom, Inc. All rights reserved. MAT-XXXX